



## **Cystatin C Calibrator**

Cat. No.: 51240-S1

Type: Calibrator

### **KIT COMPOSITION**

Calibrator 1-5      5 x 1ml

### **INTENDED USE**

The calibrator is intended to be used for the calibration of the *ImmunoDiagnostics Limited* Cystatin C assay (Cat. No. 51240)

### **CONTROL PREPARATION**

The calibrators are ready to use and do not require reconstitution.

### **STORAGE AND HANDLING**

All calibrators should be stored refrigerated (2-8°C). Return all calibrators to 2-8°C promptly after use. Unopened calibrators can be used for one year from the date of manufacturing as indicated on the expiration date on the package and bottle labels. Opened bottles of calibrators can be used for 1 month if stored at 2-8°C.

### **CALIBRATOR VALUES**

Calibrator 1: 0.5 mg/L

Calibrator 2: 1.0 mg/L

Calibrator 3: 2.0 mg/L

Calibrator 4: 4.0 mg/L

Calibrator 5: 8.0 mg/L

