



α 1-Microglobulin Control

Cat. No.: 51900-C1

Type: Control

KIT COMPOSITION

Control 1-2 2 x 1ml

INTENDED USE

This control is intended to be used as a consistent test sample of known concentrations for monitoring the *ImmunoDiagnostics Limited* α 1-Microglobulin assay (Cat. No. 51900)

CONTROL PREPARATION

The controls are ready to use and do not require reconstitution.

STORAGE AND HANDLING

All controls should be stored refrigerated (2-8°C). Return all controls to 2-8°C promptly after use. Unopened controls can be used for one year from the date of manufacturing as indicated on the expiration date on the package and bottle labels. Opened bottles of control can be used for 1 month if stored at 2-8°C.

CONTROL VALUES

Control 1: 10.8~13.2 mg/L

Control 2: 45.0~55.0 mg/L

